

## EQUIPMENT

# OR Light Preventative Maintenance

### ***Preventative Maintenance:***

- Checking mechanical functionality (grease, tightening, etc.),
  - Check supporting framework for mobility
  - If framework is high resistive to movement, grease resistive parts and/or loosen braking screws
  - If framework moves too easily (does not stay in place after adjustment), tighten braking screws
- Cleaning bulb housings
  - Clean the bulb housings periodically so lengthen bulb life and improve light quality
  - Do not touch IR filters with bare hands
  - Fingerprints left on bulbs/filters will cause overheating and could damage lamp
- Keep any bulbs that could be viable replacements available
  - Salvage functional light bulbs from non-functional OR lamps and keep for future use as replacements in other lamps
  - Keep any bulbs that are viable replacements for those in lamp to use as replacements when needed