

## EQUIPMENT

# Oxygen Concentrator Preventative Maintenance

### ***Preventive Maintenance***

*Always run machine for a few hours after maintenance before patient use*

1. Clean humidifier and tubes
  - a. Wash in warm soapy water, rinse thoroughly, and replace
2. Change distilled water
3. Remove and clean the foam air intake filter
  - a. Wash with soap and water
  - b. Ensure filter is dry *before* replacing but *do not* use heat to dry
4. Check Alarm system battery
  - a. An audible alarm should sound when the machine is turned on
  - b. Replace 9V battery when necessary
5. Check if Zeolite Canisters have expired (25,000 hours). The granules become grey when they are no longer effective.
6. Felt pre-filter should be changed once a month (if available)
7. Replace patient bacterial filter annually (if available)