

Infant Warmer Preventive Maintenance

1. Check for signs of physical damage or abuse
2. Check for evidence of fluid spills
3. Check for secure mounting
4. Check casters/brakes/mounting
5. Check AC plug/cord/receptacle
6. Check strain relief at both ends of cord
7. Check controls/switches
8. Check caution/operation labels present & legible
9. Check filters. Clean/replace as needed
10. Check power-on sequence
11. Test all audible & visual alarms and indicators
12. Clean interior/exterior as required
13. Test HI Temp Alarm
14. Measure Temperature at various control settings
15. Measure chassis ground resistance
16. Measure chassis leakage current
17. Check battery
18. Cleaned, Disinfected between uses
19. Clean skin temp sensor
20. Check for corrosion and tightness of the heating element connections